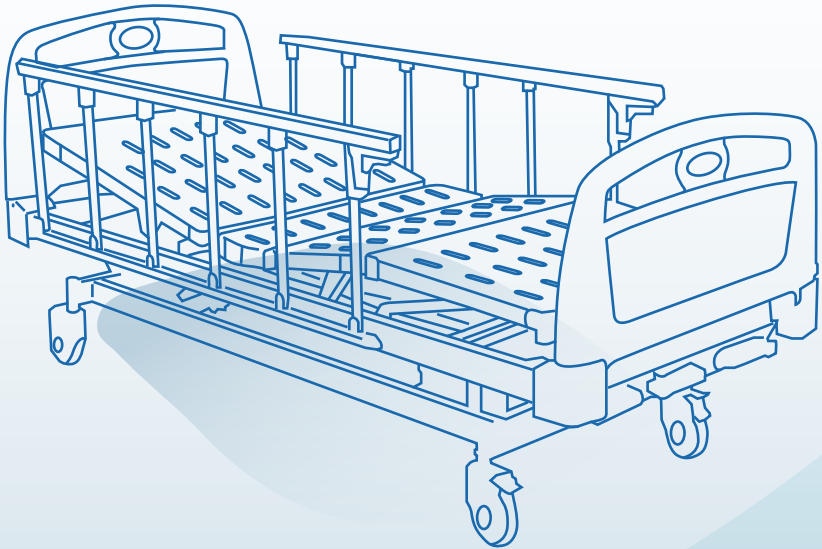
HOSPITAL BED

Katil Hospital

CONTENT / ISI KANDUNGAN

I. Introduction / Pengenalan	1
II. Parts of the Hospital Bed / Bahagian Katil Hospital	2
III. Method of Usage and Operation / Kaedah Penggunaan dan Operasi	3
IV. Safety precaution when using the Hospital Bed / Langkah-Langkah Keselamatan Apabila Menggunakan Katil Hospital	4
V. Explanation of Product Label / Penjelasan Terperinci Mengenai Label Produk	4
VI. After-sales service Product Consultation Card / Kad Produk Perkhidmatan Konsultasi Selepas Pembelian	5
VII. Online Warranty Registration / Pendaftaran Waranti Dalam Talian	6

I. INTRODUCTION / PENGENALAN

Thank you for choosing Felco product. We provide different hospital bed according to individual requirement and clinical assessment aspect. Its operation is easy, convenient and simple.

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It can be used by hospitalized patients, patients and advanced-age and poor health patient.

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We have developed multiple models of hospital bed according to different usage environment and various user's requirement. Our hospital bed are made in high-quality materials and provided with special surface treatment, which are secured and durable and have a prolonged service life. The wheel of the hospital bed is made of superior material, which is wear resistant and has no smell, quiet and stable, make customer use with comfort and relief.

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Intended of Use : Hospital beds is intended for use at home. It is intended to assist hospitalized patients and other in need of some form of health care. It is a class A medical device. Hospital beds have special features both for the comfort and wellbeing of the patient and for the convenience of the health care workers.

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Contradictions : This type of device must not be used by a disabled person or hospitalised patient alone. The user should always accompanied by a caregiver or assistance person. For heavy user require special bed for extra stability and support of the user's body. The need for this type of hospital bed should be always estimated and indicated by a physician or a physical therapist.

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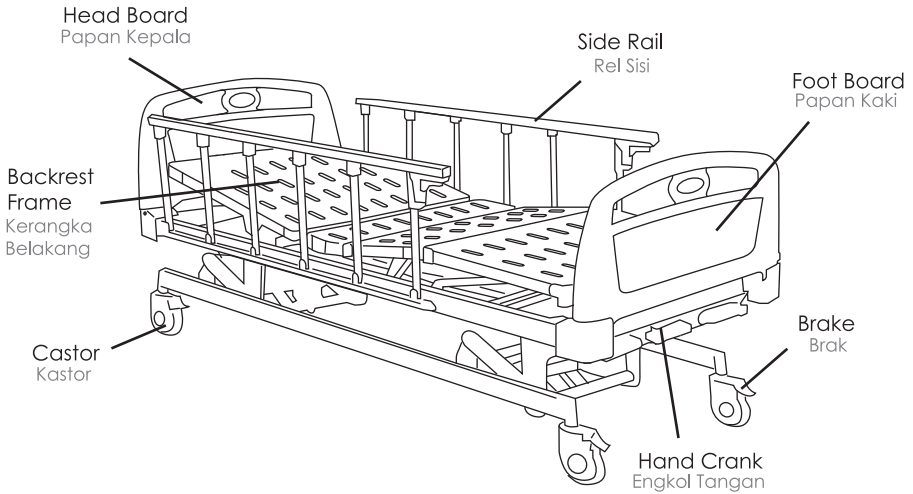
Be careful fingers or other body parts of user and/ or assistant can be trapped between the device's part during adjusting of the device. Be extremely careful when adjusting the device. Observe the dimensions of gaps between the device part's to avoid trapping your fingers or other body parts.

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
II. PARTS OF THE HOSPITAL BED / BAHAGIAN KATIL HOSPITAL

FMB-XXX serial Hospital Bed have 2 type; Electric Hospital Bed & Manual Hospital Bed

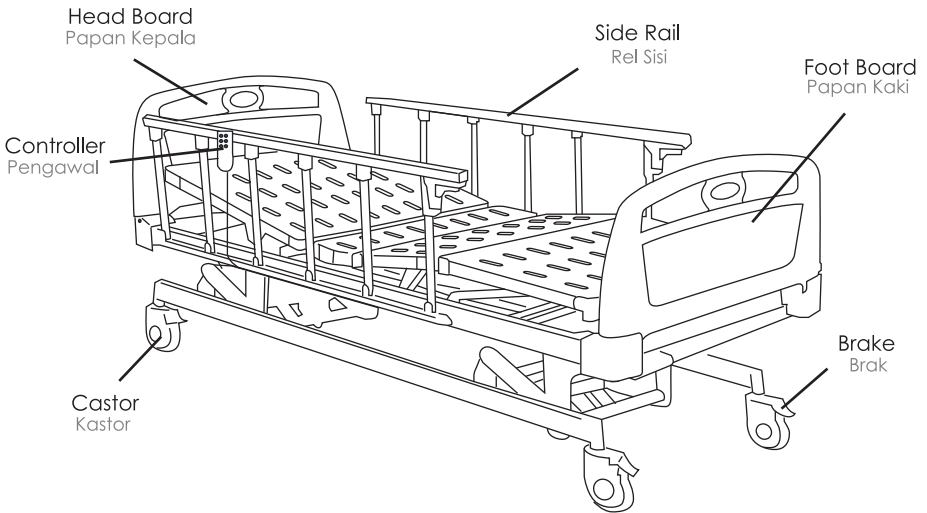Manual Hospital Bed / Katil Hospital Manual

Electrical Hospital Bed / Katil Hospital Elektrik

III. METHOD OF USAGE AND OPERATION / KAEDAH PENGGUNAAN DAN OPERASI

1) Lowering and raising of bed side rail.

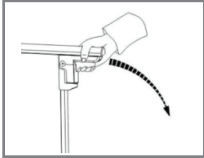
To lower the side rail : Squeeze the handle and pull it backwards.

To raise the side rail : Hold the handle and push forward until it clicks into place.

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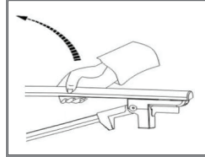
Raising
Menaikan

Note :

Do not put your hands on the joints of the side rail of the bed when lowering or raising the side rail.

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2) Method of usage for crank.

Pull out the entire handle and turn the handle.

To lower the position : Turn to anticlockwise

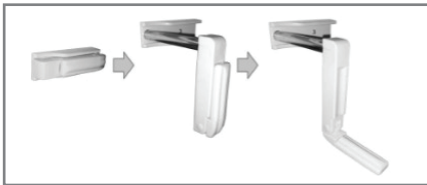
To raise the position : Turn to clockwise

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Note :

Do not overload use of hospital bed.

Lock the castor when not moving the bed.

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Lower the back & leg
Turun belakang & kaki

Note :

Do not use the controller with wet hands, do not wet the controller and the motor

Do not drop or place heavy objects on the controller.

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IV. SAFETY PRECAUTION WHEN USING THE HOSPITAL BED

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**MAX CAPACITY
100KG ONLY**

PINCH POINT

After-Sales Service Product Consultation Card

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In order to better ensure product quality and improve product design, and to further meet customer demand, please submit your valuable opinions to our factory.

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Online Warranty Registration / Pendaftaran Waranti Dalam Talian

SCAN ME

- 1) Scan QR code.
- 2) Fill up your name, contact number, email.
- 3) Select product type from :
 - i. Thermometer
 - ii. Commode
 - iii. Wheelchair (Manual)
 - iv. Wheelchair (Electrical)
 - v. Hospital Bed (Manual)
 - Vi. Hospital bed (Electrical)
 - Vii. Blood Pressure Monitor
 - Viii. Nebulizer
 - Ix. Weighting Scale
 - X. Others
- 4) Fill in purchase date according to purchase receipt date.
- 5) Select purchase from :
 - i. Pharmacy
 - ii. Clinic
 - iii. Hospital
 - iv. Official Store
 - v. Other
- 6) Fill in product serial number.
- 7) Upload receipt or warranty card in jpeg/png/pdf format.
- 8) Submit the form and you will receive an email copy.

SERVICES:

INSTALLATION

**REPAIR &
MAINTENANCE**

DELIVERY

CONSULTATION

NO 09, JALAN SERENDAH 26/41, SEKITAR 26, 40400 SHAH ALAM, SELANGOR, MALAYSIA
TEL: +603-5525 1225 & 5525 1228 FAX: +603-5525 2223 SERVICE DEPARTMENT: +6012-315 9223
CARE GDPMD CERTIFICATE NO: MYG2194707

MEDICAL DEVICE AUTHORITY MALAYSIA, LESEN NO: MDA-1615-WDP121